

# Portable Ultrasound Scanner V12

## Quick Guide

### Your ultrasound scanner kit contains:

- 1x Main unit (ultrasound scanner)
- 1x Rechargeable battery
- 1x Silicone case with belts to protect your ultrasound scanner
- 1x Power adapter 100 to 240 volts
- 1x Probe
- 1x Carrying case

---

### Quick guide to start using your Ultrasound Scanner:

#### First Step: Recharge battery and connect the probe.

Before you turn on your ultrasound you will need to connect the probe to the main unit otherwise the system will not recognize the probe.

We suggest that you recharge the battery for at least 3 hours before first use. You can use the ultrasound machine while you recharge the battery using the power adapter.

#### Second Step: Turn on the main unit and press 7.

Once you have connected the probe and have recharged the battery you can turn on the ultrasound by pressing the Power button located on the right side (same side where the probe port is located).

Press (7) after you see "Welcome" on the screen to start using the ultrasound.


#### Notes:

1. Remember that you need to apply ultrasound gel on the animal otherwise you won't see any images on the screen.
2. You need to have experience on reading ultrasound images otherwise you won't understand what you see.
3. Your V12 comes with a mechanical convex probe that is perfect to test pregnancy on dogs, cats and pigs. Convex probes are for abdominal examination not for rectal examination.
4. You need to wait between 25 to 34 days before you can confirm pregnancy and count puppies.

# Portable Ultrasound Scanner V12

For veterinary use only

## Overview

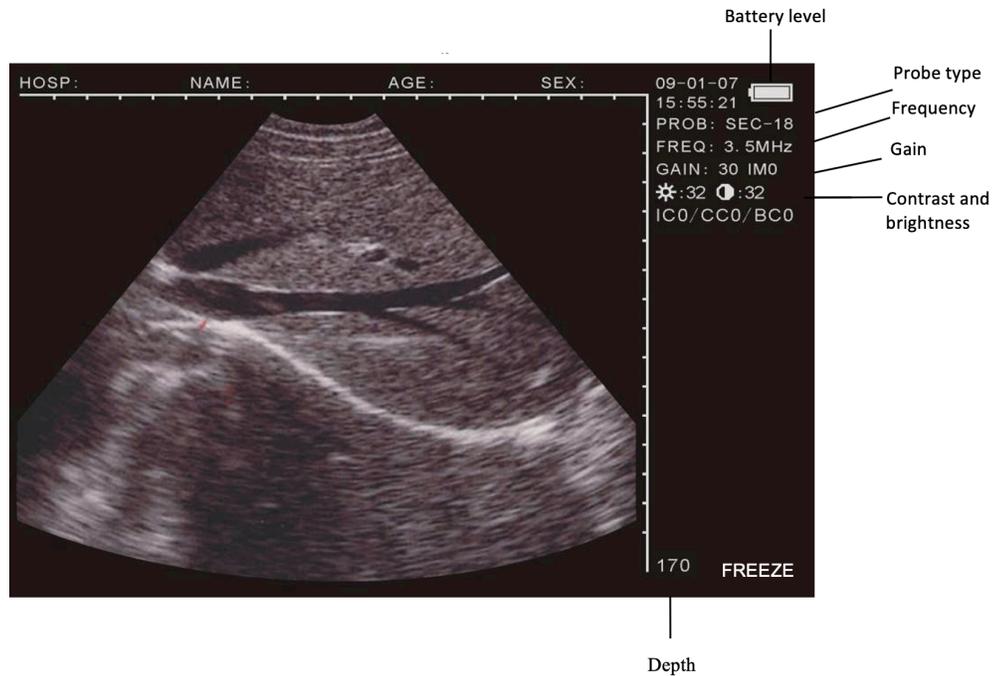

### Quick guide to adjust settings and enter information using the keyboard:

- Press **(5)** or **(4)** to select frequency, brightness, contrast or gain then press **(3)** or **(4)** to adjust.
- Press **(1)** to adjust Depth. Usually 160 depth is the best.
- Press **(Freeze)** to freeze or unfreeze the image.
- Press **(M)** or **(0)** to see more options then select the option you want using the numbers in the keyboard. For example, press **(0)** to save an image, **(1)** to see a saved image.
- Press **(Freeze)** then **(MENU)** to see more options like “restore factory settings” or to enter: Name, Age, Sex, Time, etc.

---

**Download the complete user guide at ValueStore.us**

**Visit: [ValueStore.us/download](http://ValueStore.us/download)** then click on: Ultrasound Scanner V12.

**Questions?** Visit ValueStore.us or email us at [customercare@valuestore.us](mailto:customercare@valuestore.us)